
INDESmed

Improving your future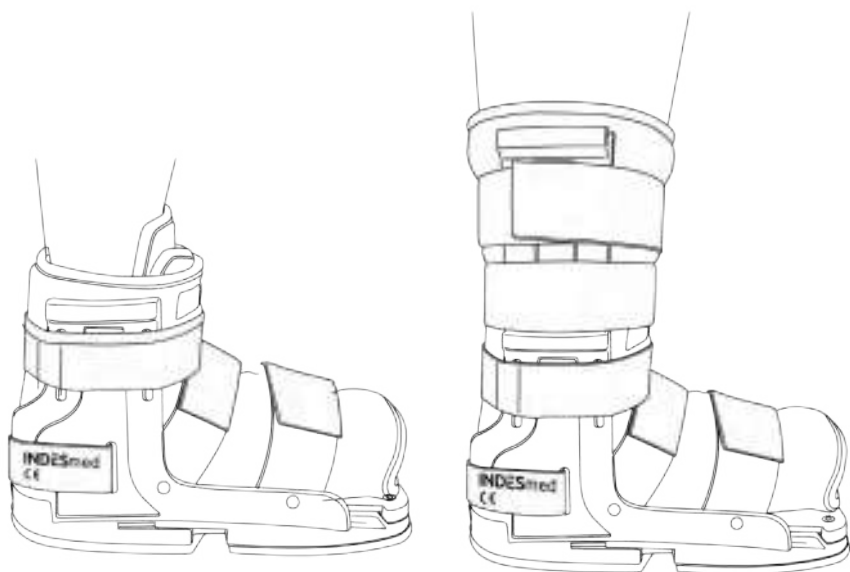

Manual de ajuste y uso BOTA WALKER INDESmed

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User manual INDESmed Walker boot

The INDESmed walker boot is a unique product, patented worldwide, that allows a perfect fit to the patient's foot during the different stages of the recovery process.

It can be used as a short boot (default), or as a long boot using the extenders. Valid for sizes 37 to 46. Maximum load: 120 kg


Indications

Limitation and functional restriction and movement of the foot and ankle, in therapeutic processes and post-surgical recoveries such as: acute ankle sprains, soft tissues ankle sprains, soft tissue injuries of the calf, tibia and fibula stress fractures, stable fractures of the foot and ankle, stable distal fractures of the tibia and fibula. This product has been designed to complement the medical treatments to the above pathologies.

*Not indicated for patients with an EU shoe size lower than 37 or with an EU size higher than 46, patients with hypersensitivity of the skin or diseases related to coagulation. Consult a doctor in these cases.

Contraindications

In fractures not proven to be stable and when there are no acceptable limits of angular and rotational deformity. Do not use the product on open wounds.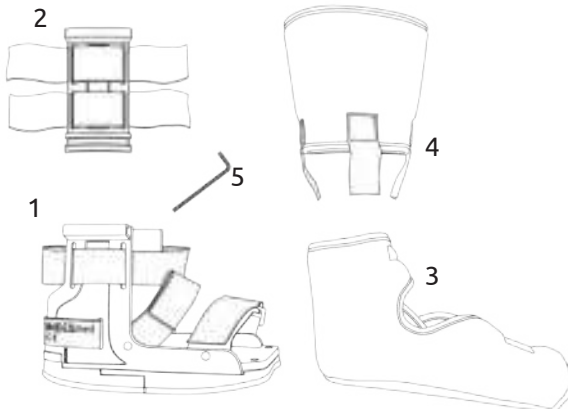

Contents of the box

1. Short Walker boot structure with straps.
2. Lateral extensions for long/high Walker boot with straps.
3. Inner lining for short boot.
4. Inner lining extension for long boot.
5. Allen wrench.

SIZE SETTING

Before fitting your boot:

Adjust the estimated length of the boot using the allen wrench at the heel adjustment point (Figure 3). Refer to Table 1 to determine the length of the boot based on your foot size.

To lengthen the boot, turn the allen wrench counterclockwise; to shorten the boot length, turn the allen wrench clockwise.

The boot will adjust the width of the foot automatically (Figures 1 and 2).

Caution: Follow the instructions of your doctor for a better fit.

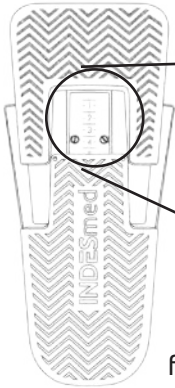
fig. 1

fig. 2

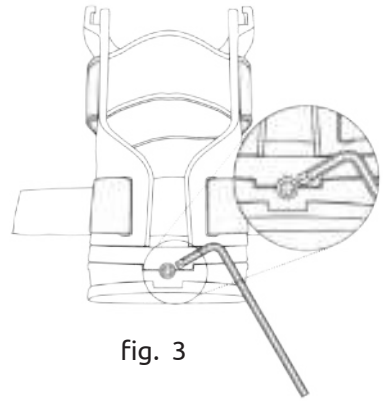
fig. 3

INDICATIVE SIZE CHART

Adjustment scale cm EU foot size

0	23,5	37
1	24,5	38
1,5	25,5	39
2	26	40
2,5	27	41
3	27,5	42
3,5	28,5	43
4	29	44
4,5	30	45
5	30,5	46

IMPORTANT: Always follow your doctor's instructions. The correct fitting and adjustment of the INDESmed Walker Boot must be supervised by a qualified health professional or orthoprosthesis Technician. If excessive pain or discomfort occurs during the use of the product, ask for help from your doctor or health care technician immediately.

fig. 4

fig. 5a

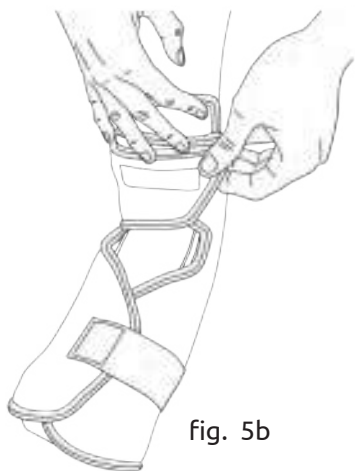
fig. 5b

fig. 6

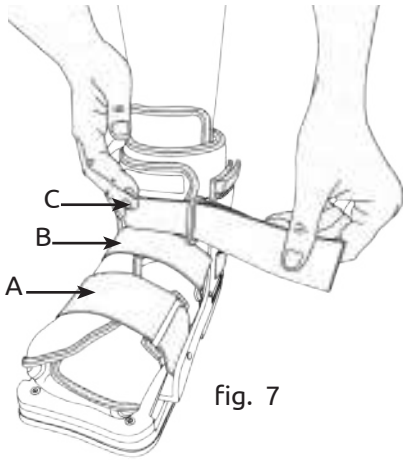
fig. 7

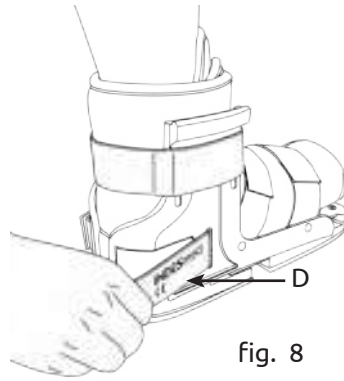
fig. 8

fig. 9

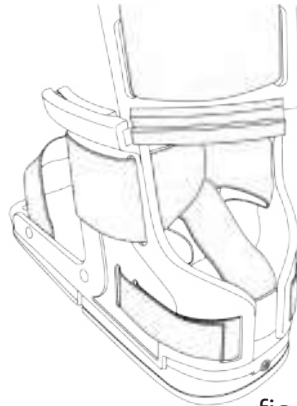
fig. 10

fig. 11

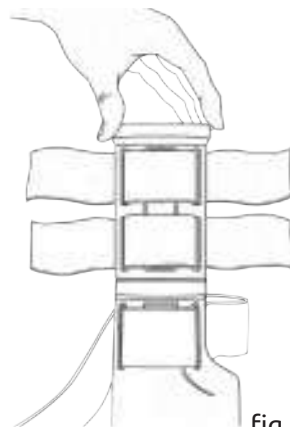
fig. 12

fig. 13

fig. 14

fig. 15

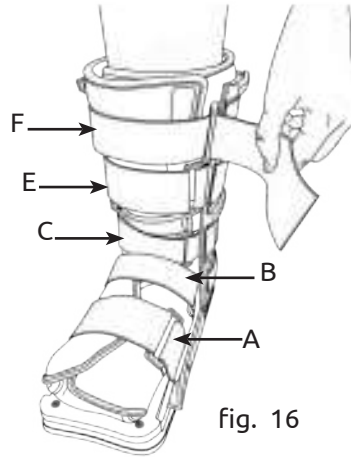
fig. 16

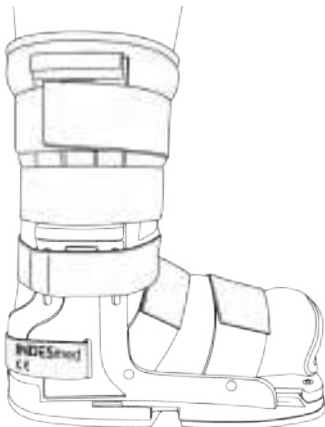
fig. 17

HOW TO FIT THE BOOT

SHORT WALKER BOOT

- 1- Have a seat to put on the boot.
- 2- Open the protective inner lining, insert the foot and place the flap along the instep of the foot (Figure 4).
- 3- Fold the flaps of the protective inner lining, starting with those of the instep of the foot, adjusting them with the straps (figure 5a and figure 5b).
- 4- Insert the foot, with the protective inner lining tightened, into the boot and check that the size is correct; your heel should reach the end of the boot (figure 6). If this is not the case, readjust the boot according to section 2. (Size adjustment)
- 5- Then proceed to adjust or tighten the straps (A, B and C): two over the instep of the foot (A, B) and one over the ankle (C) (figure 7). Tighten the heel strap (D) (figure 8).

Check the correct immobilization of the ankle, which should be supported avoiding flexion-extension or adduction movements of the foot (Figure 9).

If necessary, repeat the adjustment of the straps (Figures 7 and 8), and adjust the size of the boot (Section 2. Size adjustment) if you feel that there is an excessive movement.

LONG WALKER BOOT

- 1- Have a seat to put on the boot.
- 2- Attach the two extensions to the base of the short boot, sliding them along the rails of the side connection area until both are joined and locked together. (Figures 10-11-12)
- 3- Open the protective inner lining, insert the foot and place the flap along the instep of the foot (Figure 4).
- 4- Fold the flaps of the protective inner lining, starting with those of the instep of the foot, adjusting them with the straps (figure 5a and figure 5b).
- 5- Add the protective inner lining extension to the short one by adjusting the three velcro straps to join the two pieces. (figure 13 and 14)
- 6- Insert the foot, with the protective inner lining tightened, into the boot and check that the size is correct; your heel should reach the end of the boot (figure 6). If this is not the case, readjust the boot according to section 2. (Size adjustment)
- 7- Then proceed to adjust or tighten the straps (A, B, C, E and F): two over the instep of the foot (A, B), one over the ankle (C) and two on the lateral extensions (E and F).

Tighten the heel strap (D) (figure 8).

Check the correct immobilization of the ankle, which should be supported avoiding flexion-extension or adduction movements of the foot (Figure 17).

If necessary, repeat the adjustment of the straps (Figure 16), and adjust the size of the boot (Section 2. Size adjustment) if you feel that there is an excessive movement.

CARE INSTRUCTIONS

Hand wash both the boot and the protective inner lining.
The sole can be cleaned with warm water and a conventional detergent.
After each use, check that no foreign bodies are lodged in the sole opening.

For the protective inner lining:

- Hand wash the product with a neutral detergent.
- Do not bleach.
- Air dry.
- Do not iron.
- Do not dry clean.

Soap residues may cause skin irritation and can damage the fabric.

When not in use, store the product in a dry place protected from sunlight.

WARRANTY

INDES medical S.L. guarantees all its products, as long as they have not been manipulated or altered in their original configuration.

It does not guarantee those products in which by misuse, deficiencies or breakage of any kind, their characteristics are affected.

Warning:

This medical product is only to be used on one patient. In the case it is used for the treatment of more than one patient, the manufacturer's responsibility for the product disappears.

USER INFORMATION

For an effective / safe use of the product:

Read the instructions for use of the product.

Read the product's contra-indications.

It is recommended to previously consult a doctor for supervision and/or pre-registration.

Non-sterile device.

At the end of its lifetime, deposit the product in the recycling containers.

Do not expose to extreme temperatures. Keep between temperature limits from -30 to 50°C.

Store in a non-condensing environment, between 0 and 90% of ambient relative humidity.

In case of observing a malfunction of the device, notify the manufacturer as soon as possible: info@indesmed.com. The manufacturer will proceed accordingly.

Any serious incident should be reported to INDES MEDICAL S.L. as well as to the National Competent Authority of the country.

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